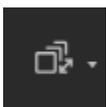
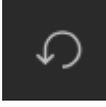


Image Viewer Toolbox Legend

Icon	Icon Name	Description
	Window Level	Enables you to manually adjust the window width and level. Width can be wide (many grays, less contrast) or narrow (fewer grays, more contrast). Level can be high (dark) or low (bright).
	Interactive Zoom	Zooms the selected image. In the viewport, click and hold the left mouse button, and move the cursor up/down to zoom in/out on the center of the image.
	Pan	Repositions the image within the viewport.
	Line Measurement	The application displays the distance between the starting point and ending point in millimeters when the image is calibrated. When not calibrated, it displays in pixels (px).
	Scroll Tool	Enables you to scroll through the images in a series within a viewport by moving the mouse cursor up or down while the left mouse button is pressed.
	Angle	Click three points on the image. The application displays the angle's measurement. You can also modify any existing angle measurements. Click and hold an endpoint, drag to the desired location, and release.
	Annotation Ellipse	Click and drag the mouse cursor to draw an ellipse over the desired region. When you release the mouse button, the application displays the ROI measurements.
	Image Orientation	Rotates the image clockwise in 90-degree increments.
	Image Orientation	Flips the image on the horizontal axis.
	Pixel Value	Identifies the pixel coordinates and the pixel value of a pixel when you click on an image and hold the button. The pixel value is expressed as an RGB value for color images or a grayscale value (representing brightness) for monochrome images.
	Add Text	Click the image where you want the text to appear. The application places a text insertion point on the image. Type the text you want to place on the image. Click outside of the text box or press Enter to complete the function.
	Reset	Resets your presentation to the initial state.